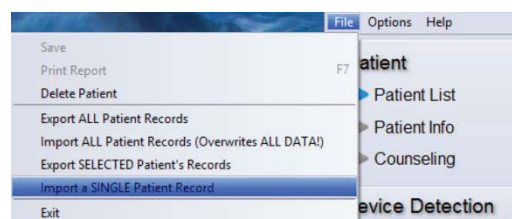
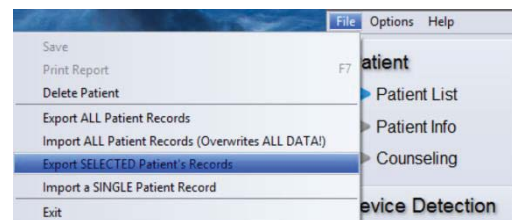
SOPHONO™ TRANSFER PATIENT DATA

How to transfer one patients data from one computer to another

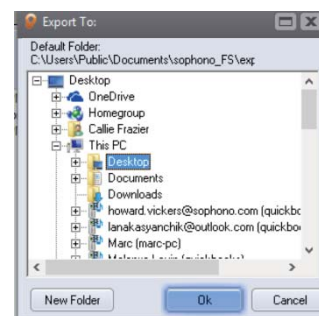
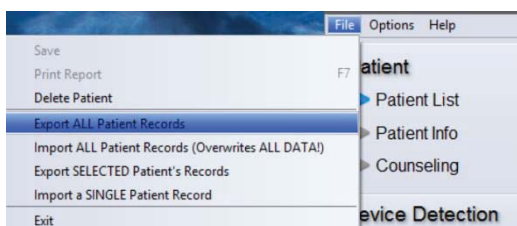
1. Open the Fitting Software on the old computer.
2. From the Patient List screen, go to File > export SINGLE patient record.
3. Save the file to a USB memory stick.
4. Move the USB stick from the old computer to the new computer.
5. Open the Fitting Software on the new computer.
6. From the Patient List screen, go to File > Import SINGLE patient record.
7. Select the patient record from the USB memory stick.

How to transfer all patient data from one computer to another



Old Computer

1. Open the Fitting Software on the old computer.
2. From the Patient List screen, go to File > export ALL patient records.
3. Save the file to a USB memory stick.
4. Move the USB stick from the old computer to the new computer.

New Computer

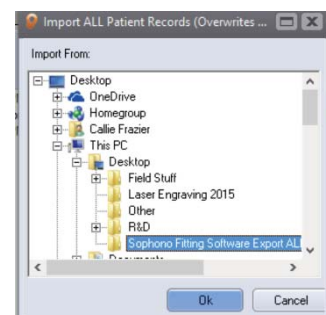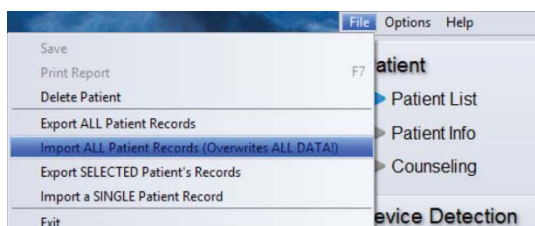1. Transfer the folder to the desktop of the new computer.
2. Open the Sophono Fitting Software and go to File > Import ALL Patient Records
3. Select the Folder from the desktop and select OK.
4. All patients will be imported into the software. However, repeat patients will not be overwritten.

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© 2017 Medtronic, Inc.