
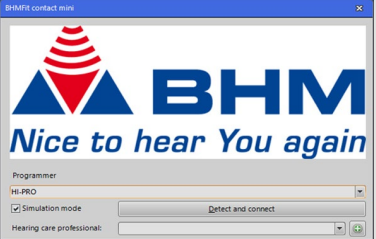


1 Connect 4-pin programming cable to HI-PRO interface and to programming socket of contact mini which is located under the housing cover. Programming without or with battery possible!




4-pin programming socket


2 Start program. (BHMFit contact mini)



3 Open customer or create „New Customer“ and enter audiogram. (you can also use in-situ function see SW user instruction) Open session or create „New session“.

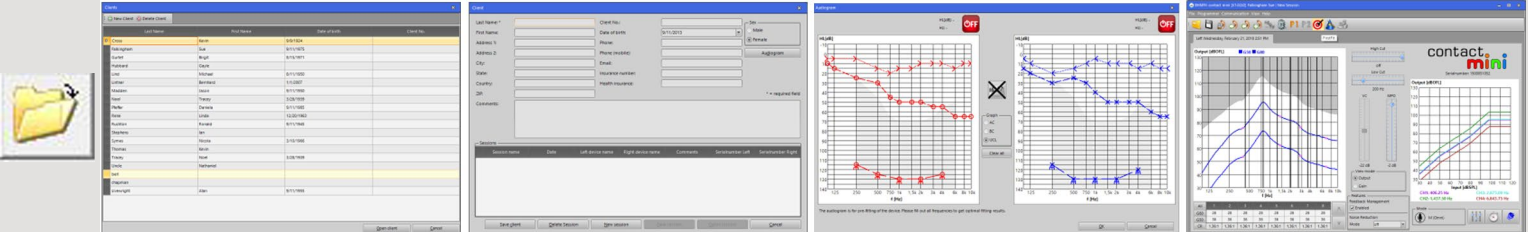


4 Press „First-Fit“. (Button)




5 Adjust gain - you can select each field, column and row.

All	1	2	3	4	5	6	7	8	^
G80	28	28	28	24	24	24	24	24	
G50	40	40	40	36	36	36	36	36	v
CR	1.67:1	1.67:1	1.67:1	1.67:1	1.67:1	1.67:1	1.67:1	1.67:1	

6 Adjust Low Cut. (activated Low Cut improves speech understanding and increases lifetime of battery)

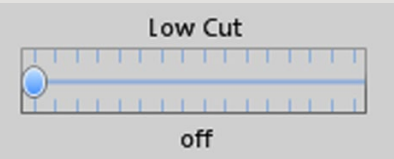

7 Adjust MPO. (activated MPO avoids uncomfortable loud noise)



8 Store setting in the aid and save session when setting is perfect.



9 Exit program by „File - Exit“. Disconnect programming cable and mount housing cover carefully.