
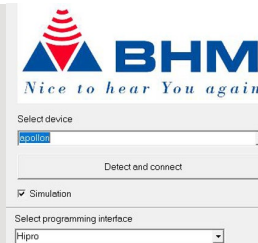



1 Insert AA batteries in your hearing aid. Connect 4-pin programming cable to HI-PRO interface and to programming socket of apollon which is located under the shirt clip. Switch on the device.




2 Start program. (BHMFit2)



3 Select receiver type and side of fitting.

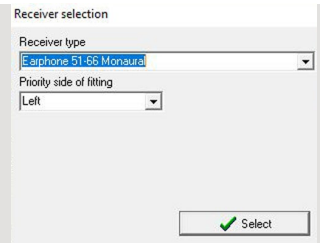

4 Open customer or create „New Customer“ and enter audiogram. Open session or create „New session“.

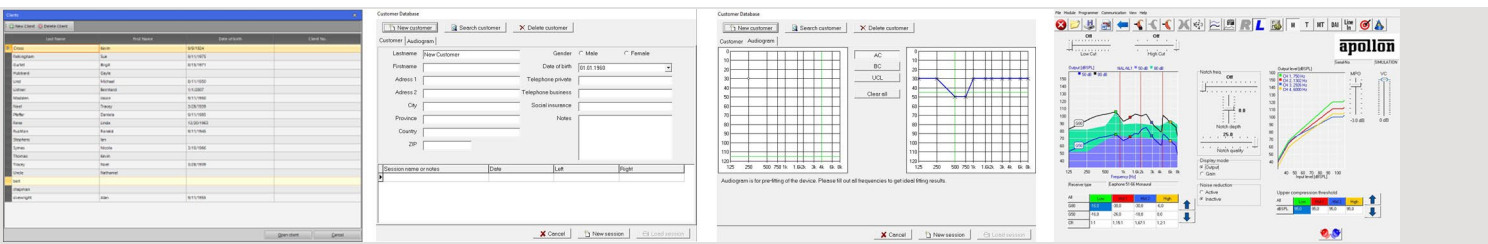


5 Press „First-Fit“. (Button)




6 Adjust gain - you can select each field, column and row.

All	Low	Mid 1	Mid 2	High
G80	-16,0	-30,0	-30,0	-6,0
G50	-16,0	-26,0	-18,0	0,0
CR	1:1	1,15:1	1,67:1	1,2:1


7 Adjust MPO. (activated MPO avoids uncomfortable loud noise)




8 Copy program and transmit settings to all modes.

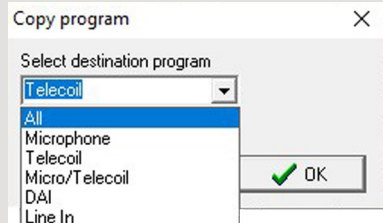

9 Store setting in the aid and save session when setting is perfect.



10 Exit program by „File - Quit“ or press „Quit“. (Button)  Disconnect programming cable and mount shirt clip carefully.